Ritter M9[®]/M11[®] Loading Cassettes

Below are recommended cassette load configurations for the M9/M11. For complete instructions, see the M9/M11 User Guide.



M9 CONFIGURATIONS							
Pouch Size	Cassette Size	Trays	Pouch Racks	Pouches/Load	Cassette/Load	Load Configuration	Example
N/A	medium	1 small	0	0	2	Slot 1 (top): empty Slot 2: empty Slot 3: empty Slot 4 (bottom): small tray, two cassettes	
9" x 3.5"	medium	2 small	1	5	1	Slot 1 (top): small tray, cassette Slot 2: empty Slot 3: empty Slot 2: small tray, pouch rack	
M11 CONFIGURATIONS							
Pouch Size	Cassette Size	Trays	Pouch Racks	Pouches/Load	Cassette/Load	Load Configuration	Example
9" x 3.5"	small	1 small 1 large	1	6	3	Slot 1 (top): empty Slot 2: large tray, 3 cassettes Slot 3: empty Slot 4 (bottom): small tray, pouch rack	
N/A	large	0	0	0	4	Vertical cassette rack with 4 large cassetees	
N/A	large	0	0	0	4	Horizontal cassette rack with 4 large cassettes	
N/A	large	0	0	0	4	Horizontal cassette rack with 4 large cassettes	

🛆 Always use pouch racks on the bottom tray. Stand cassettes in the bottom tray on edge. Do not place cassettes flat on the bottom tray.